

# Shanoir Tutorial for Experts

---

*Shanoir version 0.4*



**Bienvenue dans Shanoir**

Nom d'utilisateur

Mot de passe

Se souvenir de moi

☐

**Connexion**

[Vous avez oublié votre mot de passe?](#)



[Demander un compte](#)

Copyright 2012 

## Contenu

View and Manage members of my Research Study .....	2
Delete Data .....	3
Delete Datasets .....	3
Delete Examination .....	4
Delete Subject .....	5

## View and Manage members of my Research Study

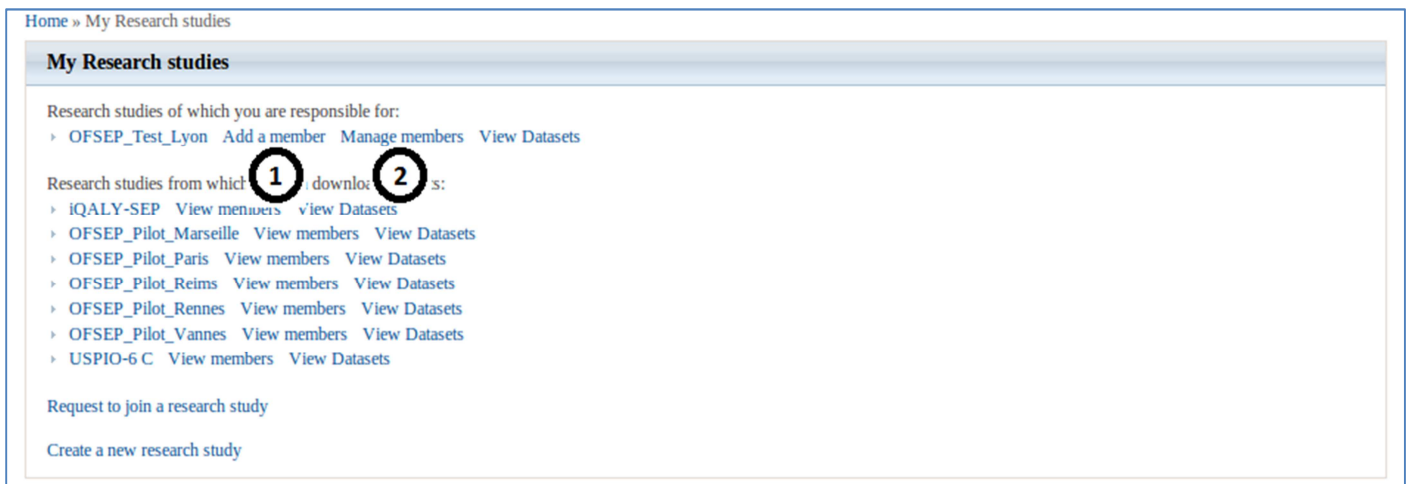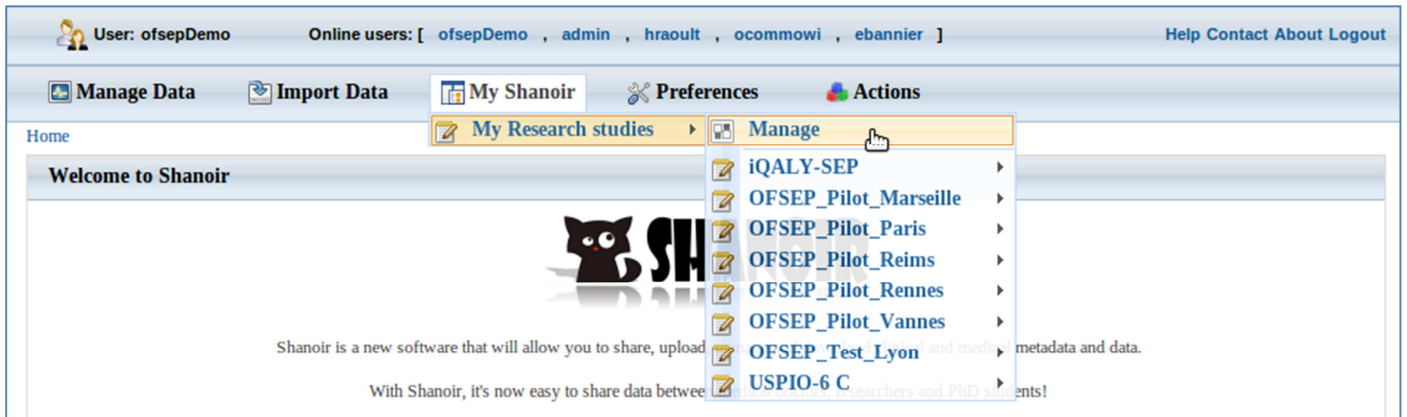

1) Add a member

**Warning :** the “Add member box” appears at the bottom of the page, click and scroll down

2) Manage members

- Change role/position
- Change the email notification (import or anonymization)
- Delete members

## Delete Data


### Reminder :

- A study contains examinations
- A subject is link to his examinations
- An examination contains datasets

You can delete datasets, examinations, subjects.  
But you can't delete something that is not empty.  
That is to say that you can't :

- Delete an Examination that contains Datasets
- Delete a Subject that has Examination

## Delete Datasets

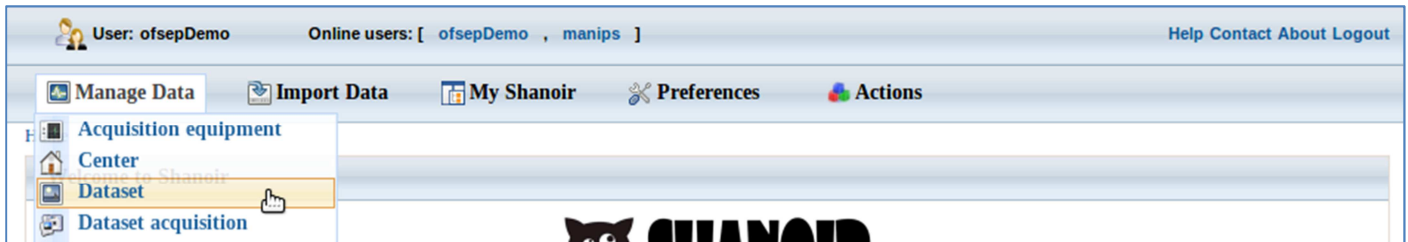

Home » Manage Datasets

### General criteria

Study name: \* 1

OFSEP\_Test\_Lyon 2

☒ Result of ☐ Data of

Dataset name: 3

Examination comment:

Dataset creation date:

From  To

Subject name: 06015HUBY 4

Advanced Search Criteria >>

**Search Datasets** 5

Your selections: res. EP\_Test\_Lyon - subject name like 06015HUBY

Dataset list								
Id	Name	Type	Subject	Is result of	Creation date	Comment	Action	Select all
72504	tse_vfl_WIP607	Mr	06015HUBY	OFSEP_Test_Lyon	04/03/2010	tse_vfl_WIP607		<input type="checkbox"/>
72503	tse_vfl_WIP607	Mr	06015HUBY	OFSEP_Test_Lyon	04/03/2010	tse_vfl_WIP607		<input type="checkbox"/>
72502	tse_vfl_WIP607	Mr	06015HUBY	OFSEP_Test_Lyon	04/03/2010	tse_vfl_WIP607		<input type="checkbox"/>

Fill the criteria of your datasets research :

- 1) Study name
- 2) Dataset name : "%" is a wildcard, it can replace any character

Ex : "%T1%" → dataset name that contains "T1" ;

"ASL%" → dataset name that starts with "ASL"

- 3) Subject name, ...

Run your research and select your data

- 4) Click on "Search Datasets"
- 5) Select the datasets you want to download :
  - Click on "Select all" or
  - Check the datasets one by one

**Search Datasets**

Your selections: result of OFSEP\_Test\_Lyon - subject name like 06015HUBY

**Dataset list**

Id	Name	Type	Subject	Is result of	Creation date	Comment	Action
72504	tse_vfl_WIP607	Mr	06015HUBY	OFSEP_Test_Lyon	04/03/2010		   
72503	tse_vfl_WIP607	Mr	06015HUBY	OFSEP_Test_Lyon	04/03/2010		   
72502	tse_vfl_WIP607	Mr	06015HUBY	OFSEP_Test_Lyon	04/03/2010	tse_vfl_WIP607	   

6) Pass the mouse on the icon above "Deselect all"

7) Click on "Delete"

## Delete Examination

User: ofsepDemo Online users: [ ofsepDemo , admin , manips ] Help Contact About Logout

Manage Data Import Data My Shanoir Preferences Actions

Acquisition equipment Center Dataset Dataset acquisition Examination Experimental group

Home » Manage Examinations


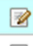





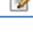

**General criteria**

Comment:  OK Notes:  OK

Examination date:

**Other criteria**

**Examination list**

Id	Subject	Research study	Examination executive	Center	Examination date	Delete	Edit	Details
2305	06015HUBY	OFSEP_Test_Lyon	François Cotton	HCL - NeuroCardio	04/03/2010			
2217	21 028	iQALY-SEP	Gilles Edan	CHU Rennes	19/03/2013			
2139	ICM_VS05	OFSEP_Pilot_Paris	Bruno Stankoff	ICM	31/07/2008			

## Delete Subject

Expert: ofsepDemo      Online users: [ ofsepDemo , admin , manips ]      Help Contact About Logout

Manage Data   Import Data   My Shanoir   Preferences   Actions

Acquisition equipment  
Center  
Dataset deleted  
Dataset acquisition  
Examination  
Experimental group  
Investigator  
Research study  
Score  
Subject creation date  
MR

Notes:  OK

OK

Home » Manage subjects

## Subject

Here you can browse, edit, search, delete and create subjects.

Create a new Subject

**General criteria**

Common name:       Subject study identifier:       Study name:

**Search Subjects**

Subject list										
Id ↑	Common name	Sex	Birth Date	Manual HD	Language HD	Imaged object category	Personal Comments	Delete	Edit	Details
1562	06015HUBY	M	01/01/1987			Living human being				
1561	0212	F	01/01/1950			Living human being				
1560	0207	F	01/01/1964			Living human being				

**Something is not clear ?**

**You have a question about Shanoir or this tutorial ?**

**Something major is missing here ?**

Please contact us !

[shanoir-contact@lists.gforge.inria.fr](mailto:shanoir-contact@lists.gforge.inria.fr)

**You want some more ?**

Ask us for our other tutorials:

“Tutoriel Shanoir pour Débutants” (french)

“Shanoir Tutorial for Beginners” (english)

“Shanoir Tutorial for Experts” (english)

“Tutoriel Study-card Shanoir” (french)

“Tutoriel ShanoirUploader” (french)